


CONTROL PANEL

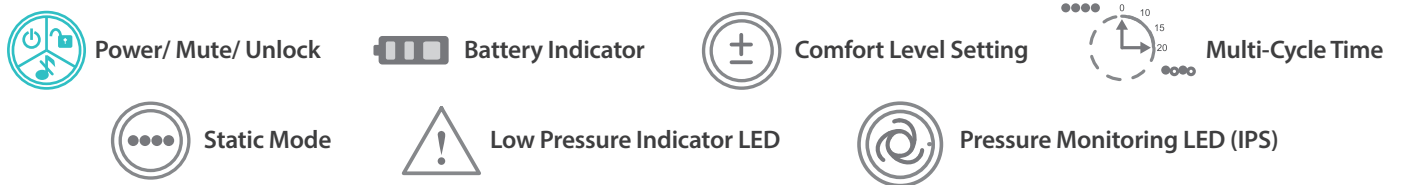

OPERATING INSTRUCTION

Power

- Press and hold for 3 seconds to power on the unit, and then the green LED will light up
- * The screen displays to indicate the device is in default Geri Chair mode

Mute

- Press to turn off the audible indicator

Unlock

The panel will auto-lock when there is no operation for 3 minutes

- Only is functional when the panel is locked
- Press and hold for 3 seconds to unlock the panel

Comfort Level Setting

- Press to adjust the setting from 1 to 5
- * The comfort setting scale is only indicative
- If the mattress seems too soft or too rigid, please adjust the comfort setting to conform to each patient's requirements

Therapy Mode Selection


Static Mode

When in Static Mode, all air cells maintain constant pressure support and maximize patient's contact area to redistribute pressure





- Press to activate this function
- Press to return to Alternating Mode
- * The green LED will light up and the Multi-Cycle Time will display 0, indicating Static Mode is activated



Multi-Cycle Time

- Press  to set up alternating time as 10, 15, or 20 minutes

Seating Cushion Mode

- Press  and  simultaneously for at least 3 seconds
- The display will change from  to , indicating the device has switched from the default Geri Chair Mode to Seating Cushion Mode



Battery Indicator

- The green LED bar indicates the battery level or charging status
- The yellow LED will light up to indicate the power unit needs to be charged for 4 hours to full battery
- * Please ensure to charge the power unit at least 8 hours before the first use




Low Pressure Indicator LED

- The yellow LED will start flashing when the cushion has insufficient pressure
- * The power unit will automatically turn off if the low pressure indicator stays for 15 minutes



Pressure Monitoring Indicator LED (IPS)

- **IPS = Intelligent Pressure Sensing** The system monitors the cushion's pressure 24 hours
- The yellow LED indicates this function is in operation

PROBLEM	INSPECT PROCEDURE	POSSIBLE SOLUTIONS
Power unit does not function	Check if it is "Low Battery"	Connect the adapter to the power outlet to charge the power unit
	Check if the orange LED is flashing and the audible indicator sounds simultaneously	Please contact your authorized local distributor for assistance
Low pressure LED is flashing during operation	Check if there is leakage in air tubes or air cells	Please contact your authorized local distributor for assistance
Power unit is working, but the cushion is not inflating	Check if cushion's air hose couplings are properly connected to power unit's quick release couplers	Secure air hose couplings firmly into place
	Check comfort level setting	Increase or decrease comfort level setting until reaching the appropriate pressure
	Inspect air filter for dust	Clean or replace air filter
	Lift the cushion coverlet to check if air cells are connected correctly	Make sure all air cells are properly connected to air supply
	Lift the cushion coverlet to check if air tubes are kinked or obstructed	Check and adjust the position of air tubes
Alternating and static setting is not available	Check if air cells are cut or cracked	Replace with genuine spare parts or contact your local distributor
	Check if the panel is locked	Press  for 3 seconds to unlock
Patient is not getting pressure relief due to system failure (reddening of skin)	Possible control failure	Please contact your authorized local distributor for assistance
		Contact your physician and / or nursing service immediately

If power unit does not respond to the possible solution, please contact your authorized local distributor for assistance