

CONTROL PANEL



ON / OFF

- Press and hold the button for 3 seconds to turn **on / off** the power unit



MUTE / RETURN

- Press for 3 seconds to mute the audible indicator
- Press to return to the previous menu
- Press two times to go back to the therapy mode selection



LIGHT FOR DISPLAY SCREEN

- Press to light on or off the display LED panel



ARROW UP / ARROW DOWN


- Press to select therapy modes and the desired pressure or operation hours



OK / UNLOCK / PAUSE

- Press to confirm the selected mode and the set values of pressure and time
- During therapy mode, Press to stop the therapy operation and press again to restart
- Press for 3 seconds to unlock the display panel

SYMBOLS ON THE DISPLAY PANEL

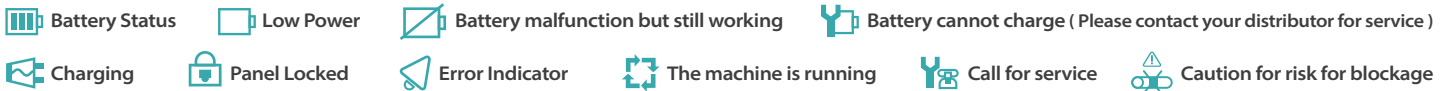

INTRODUCTION OF THERAPY MODE

- ▶ The display panel will show VT-100 when the machine is on; after 5 seconds, it will turn to the therapy mode selection page

Continuous Mode

When employed, there will be same negative pressure as selected apply all the time

- ▶ Press to move the cursor to select the therapy mode


Intermittent Mode

When employed, the therapy pressure level changes between high and low pressure range in fixed time interval settings

- ▶ Press to confirm the selection



CONTINUOUS MODE SETTING

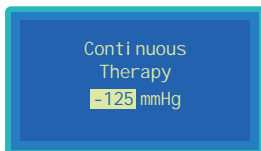

1. Press to change the constant pressure level from -30 mmHg to -200 mmHg (Default pressure for continuous therapy is -125 mmHg)
2. Press to confirm the pressure setting and start the therapy

INTERMITTENT MODE SETTING



1. Press to change the high pressure level from -50 mmHg to -200 mmHg (Default high pressure is -80 mmHg)
2. Press to confirm the high pressure setting
3. Press to change high pressure therapy time for 1 to 10 minutes (Default time for high pressure is 5 minutes)
4. Press to confirm the high pressure time setting



1. Press to change the low pressure level from -20 mmHg to -80 mmHg (Default low pressure is -20 mmHg)
2. Press to confirm the low pressure setting
3. Press to change low pressure therapy time for 1 to 10 minutes (Default time for low pressure is 2 minutes)
4. Press to confirm the low pressure time setting and start the therapy operation



NOTE 1 The display panel will be locked and turn dark when left untouched for 3 minutes

- 1 If the prescribed pressure/ therapy mode needs to be changed, press to unlock the display panel, and then press to return to setting page
- 2 During the therapy, press to see the setting details for 5 seconds

TROUBLE SHOOTING


If power unit does not respond to the possible solution, please contact Carilex authorized local distributor for assistance

ISSUE	INSPECTION PROCEDURE	POSSIBLE SOLUTION
Power Unit Does Not Function	Check if the power cord is firmly plugged into the electrical outlet	Secure power cord into the electrical outlet
	Check the battery status displayed on the display panel	Connect the power supply adapter to the electrical outlet to charge battery
	Check if the unit is still in the setting mode	Complete the settings
Insufficient Performance (The operation pressure is less than the set pressure)	Check if the tube is partially kinked or if the clip is engaged	Release clip, or remove kinks or check the system for blockages
	Check the system for leaks.	<ul style="list-style-type: none"> Connect the tubing / canister properly Seal dressings properly
	Check the battery charge status	Charge the battery
	Check whether the height of the suction pump unit is more than one meter above the wound	Move the unit within one meter height of the wound
No suction	Check if the tubing blocked	1. Release the clip, remove kinks 2. Remove clogs by applying the new dressing kit 3. Press and to activate the blockage removal function to clear the blockage NOTE 2
	Check if the Canister is full	Replace the Canister

NOTE 2


- ▶ The blockage removal function is designed to aid the removal of blockage. Due to the varying condition of individual wounds, this function may not guarantee complete and successful removal of blockage for all wounds
- ▶ The blockage removal function will create high-pressure suction, which may cause excessive bleeding. Please consult with the physician before activating the blockage removal function

ERROR INDICATORS

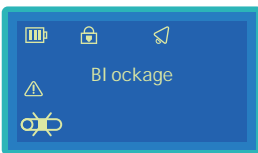


Blockage

- Press to suppress the audible indicator
- Check : (a) the tube is kinked or not, (b) the tube is straightened, (c) the clip is open, (d) the tube is clogged or not
- If the blockage indicator remains, apply the new dressing kit on the wound or press and simultaneously to activate the blockage removal function **NOTE 2**. The suction pump will run at full speed when pressing the two buttons. The process will stop once releasing the two buttons

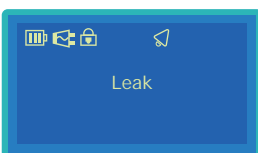


Leakage

- Press to suppress the audible indicator
- Make sure : (a) The dressing is sealed correctly, (b) The tubing is well-connected, (c) The film fully attaches to the skin to prevent leakage occurs in the dressing
- The suction pump will be back in operation mode after the problem fixed

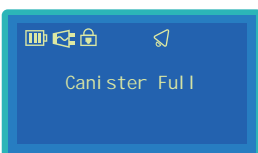


Canister Full

- Press to suppress the audible indicator
- Switch off the suction pump unit

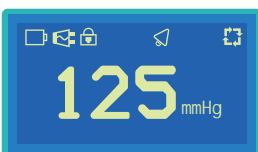

Low Battery


- Press to suppress the audible indicator
- Charge the battery

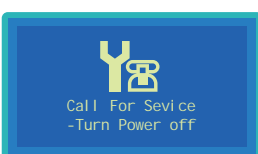

Call for Service

- Press to suppress the audible indicator
- Press to unlock the display
- Switch off the suction pump unit
- Contact the authorized local distributor for assistance

CANISTER REPLACEMENT

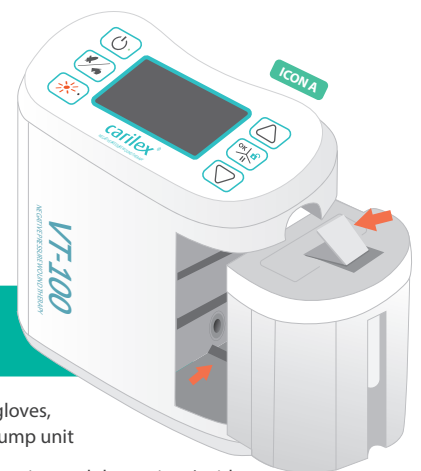

- Wash hands, wear disposal gloves, and switch off the suction pump unit
- Close the clip on both the dressing and the canister's side, then disconnect the VT-100 canister connector from the dressing
- Detach the full canister by pressing the middle button on the canister. Discard the used canister followed by local governing guidelines **ICON A**
- Insert a new VT-100 canister into the VT-100 suction pump by pressing the middle button on the canister. Make sure the bottom of the canister is aligned with the track on the device
- Installation complete when hearing a "click" sound